DKS

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■ Etchants

Select HV® Etch w/BAC, Uni-Etch® w/BAC, & Etch-37™ w/BAC	Page 7
--	--------

■ Cements

Choice 2 Kit	Page 8
Choice 2 Cement	Page 10
Veneer Cementation Technique	Page 11
BIS-SILANE	Page 12
Porcelain Etchant	Page 12
Z-Prime™ Plus.....	Page 14
ZirClean®.....	Page 15
TheraCem®.....	Page 16
Cementation Without a Bonding Agent & Zirconia Bonding Technique.....	Page 17
Duo-Link Universal™.....	Page 18
Cementation With a Bonding Agent Technique.....	Page 20

■ Composite

AELITE Aesthetic Enamel	Page 20
AELITE Aesthetic Body	Page 21
AELITE Aesthetic Posterior	Page 22
AELITE FLO	Page 23
Modeling Resin	Page 23

■ Core Materials

Core-Flo™ DC	Page 24
Core-Flo™ DC Lite.....	Page 25
Post Cementation & Core Build-up Technique	Page 26

■ Base and Liner

TheraCal LC®.....	Page 28
TheraBase®	Page 30
TheraCal PT®.....	Page 32

ALL-BOND 3®

Dual Cured Dental Adhesive

4th -Generation Bonding Agent

Total -Etching Adhesive

The gold standard in total-etch bonding just got better with BISCO's ALL-BOND 3 Universal Dental Adhesive System. ALL-BOND 3 utilizes patented technology that takes total-etch bonding to a whole new level of performance, assuring dentists of a simplified bonding process, as well as stronger, longer lasting bonds than ever before.

ALL-BOND 3 micro-mechanically bonds to all substrates, is used for all bonding procedures and is compatible with light-, dual- and self-cured materials. The ALL-BOND 3 system offers increased hydrophobicity for a long-lasting adhesive bond.

The HEMA-free, radiopaque ALL-BOND 3 Resin reduces the chance of misdiagnosing caries.

Dual Cured

No light-curing necessary when used for indirect restorations.

Low Film Thickness

part A&B 5-10 microns

Compatible

With light-cured, dual-cured, and self-cured resin cements, composites, and core materials for direct and indirect bonding procedures.

Durable

Offers increased hydrophobicity for a durable bond.

Total Etch

Used with the total-etch bonding technique.

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Light-Cured Dental Adhesive

The Universal -Generation Bonding Agent

Self Etch and Total Etch and Selective Etche

The First TRULY
Universal Dental
Adhesive

As a truly universal adhesive, All-Bond Universal is a single bottle system that is 100% compatible with all light-cured, self-cured and dual-cured material and indicated all direct and indirect procedures. It's versatile in that it can be used with any bonding technique (self-, total-, and selective-etch). The **UNIVERSALITY** of All-Bond Universal makes it an indispensable part of any dental practice

Versatile

Offers the flexibility for total-, self-, or selective-etch procedures

High Shear Bond Strengths

High shear bond strength to all substrates.

Single Bottle System

Other adhesives may need more than one bottle for indirect restorations, but with All-Bond Universal, NO activator is required

No Refrigeration Required

MDP

Contains MDP for enhanced durability

Low Film Thickness

Less than 10 microns

Total "Universality"

Compatible with all light-, self- and dual-cured resin composite and cement materials for all direct and indirect procedures

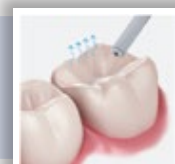
Hydrophobic formula

(resin friendly) results in improved durability of the bond

DID YOU KNOW?

All-Bond Universal's optimized formula with low water content allows for any residual water to evaporate after only 10 seconds of air-drying time.

Using All-Bond Universal: **See it in action!**

Total-etch Case

Treatment plan: lithium disilicate veneers #4-#13 with minimal preparations on enamel only.

A total etch technique was used with Etch-37 with BAC for 15 seconds and rinsed well.

All-Bond Universal was applied liberally in several coat to a moist tooth surface

All-Bond Universal was air thinned, leaving a shiny surface and then light cured for 10 second.

Choice 2 Cement was applied to the tooth surface and each veneer was placed and Light-Cured.

Veneer cementation procedure finalized.

Selective-etch Case

Final preparation before bonding procedure.

Selective etching of enamel.

Application of All Bond Universal.

Light curing of All Bond Universal.

Preparation being restored with composite placed in layers.

Final composite restoration.

Self-etch Case

Preoperative view of teeth.

Preparation for zirconia-based crowns.

Application of Z-prime Plus to intaglio surface of zirconia crown

Application of All-Bond Universal and then light cure for 10 seconds.

Application of Duo-Link Universal to the zirconia restoration.

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DUAL-Cured Dental Adhesive

The Universal -Generation Bonding Agent

Universal Primer is a dual-cured adhesive designed to be used without having to light cure the adhesive layer under indirect restorations. With chemistry based on BISCO's All-Bond Universal® technology, Universal Primer can be used with any etching restorations. With chemistry based on BISCO's All-Bond Universal® technology, Universal Primer can be used with any etching technique (self-etch, selective-etch, or total-etch) and is compatible with light-cured, dual-cured, and self-cured resin cements, composites, and core materials for direct and indirect bonding procedures.

Dual Cured

No light-curing necessary when used for indirect restorations.

MDP

Contains MDP for enhanced durability

Low Film Thickness

Less than 5µm allows adhesive to readily flow into etched surfaces.

Versatile

Can be used with total-, self, and selective-etch techniques.

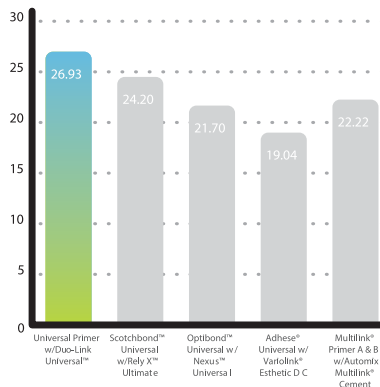
Compatible

With light-cured, dual-cured, and self-cured resin cements, composites, and core materials for direct and indirect bonding procedures.

No Refrigeration Required

Shear Bond Strength to Dentin in Self-Cure Mode (MPa)

Shear Bond Strength to Various Substrates (Duo-Link Universal)

SUBSTRATE	UNIVERSAL PRIMER
Cut Enamel (SE/ TE)	25.14 / 31.53
Dentin (SE/ TE)	26.78 / 32.47
Lithium Disilicate (silane treated)	35.02
Zirconia	31.14
Stainless Steel	35.59
Titanium	28.39
Gold	19.71
Composite	28.73

Direct and Indirect Tooth Preparation Treatment

Step 1
CHOOSE FROM
TECHNIQUE A-C.

Technique A

SELF-ETCHING TECHNIQUE:

Prepare cavity. Wash thoroughly with water spray. Use an absorbent pellet or high volume evacuation for 1-2 seconds to remove excess water (do not desiccate).

Technique B

TOTAL-ETCHING TECHNIQUE:

Prepare cavity. Wash thoroughly with water spray. Etch enamel and dentin using an etchant for 15 seconds. Rinse thoroughly. Use an absorbent pellet or high volume evacuation for 1-2 seconds to remove excess water (leave preparation visibly moist).

Technique C

SELECTIVE ETCHING TECHNIQUE:

Prepare cavity. Wash thoroughly with water spray. Syringe etchant onto cut and uncut enamel. Leave etchant in place for 15 seconds. Use high volume evacuation to remove the etchant. Rinse thoroughly. Use an absorbent pellet or high volume evacuation for 1-2 seconds to remove excess water (do not desiccate).

Step 2

Dispense an equal number of drops of UNIVERSAL PRIMER Parts A & B (1:1 ratio) into a clean mixing well. Using a brush, mix adhesive for 5 seconds. Apply two separate coats of UNIVERSAL PRIMER scrubbing the preparation with a microbrush for 10-15 seconds per coat. Do not light cure between coats.

Step 3

Evaporate excess solvent by air-drying with an air syringe for at least 10 seconds; there should be no visible movement of the adhesive. The surface should have a uniform glossy appearance; otherwise, apply an additional coat of UNIVERSAL PRIMER and air dry.

Step 4

Direct Restorations : Light cure for 10 seconds.
Indirect Restorations : Light curing is not required.
Proceed with cementation.

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Hema-Free Unfilled Resin

Hema-free Bonding agent

Porcelain Bonding Resin is a HEMA-Free, hydrophobic bonding resin designed specifically for bonding to porcelain. Due to its hydrophobic nature, this material prevents shade shifting. Placed in a thin layer on the silane-treated intaglio surface of the restoration, Porcelain Bonding Resin will act as a wetting resin allowing the cement to spread in a more uniform layer.

Porcelain Restorations

Ideal for all porcelain restorations.

No Color Shifting

Prevents additional color change

High Shear Bond Strengths

Reinforced shear bond strength to porcelain

Hema Free

HEMA-Free to make a real hydrophobic bond

Wetting resin inside of veneers

Acts as a wetting resin internal surface of veneers

CLINICAL SIGNIFICANCE

- Used for Cosmetic Facing and Eliminate any color changes
- Best Choice for bond to Enamel
- Used for intraoral porcelain repairs and bond to porcelain without bond strength reduction

Select HV® Etch w/BAC Uni-Etch® w/BAC Etch-37™ w/BAC

Phosphoric Acid Etchants with Benzalkonium Chloride (BAC)

Uni Etch 32% ,Etch-37 with Benzal Konium Chloride

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Select HV Etch is a 35% high-viscosity phosphoric acid etchant with BAC . It is specifically formulated for pinpoint placement to etch enamel when using an adhesive in selective-etch mode.

Uni-Etch w/BAC and Etch-37 w/BAC are 32% and 37% semi-gel phosphoric acid etchants with BAC. They are specifically formulated to be easily applied to larger surface areas and rinsed cleanly with no residue.

Blue Color

Easy visualization and contrast

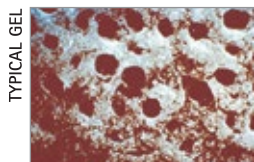
Easy Clean Up

Rinses away cleanly and quickly leaving no residue to interfere with bonding


Contains BAC


In-vitro research shows benzalkonium chloride is effective against Streptococcus mutans

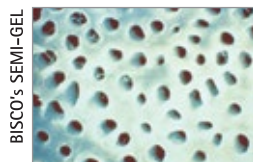TYPICAL GEL
Note the presence of silica, used as filler, on the etched, rinsed surface. Imperceptible to the naked eye, the remaining debris can impede the flow of primer over the dentin surface and into the dentinal tubules. This can compromise surface wetting and bond strength.

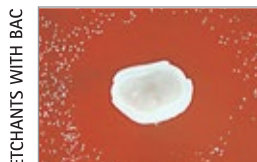BISCO'S SEMI-GEL
BISCO etchants utilize a polymer to create thixotropic properties and enhance handling characteristics. Furthermore, it leaves no residue, rinsing away completely.

ETCHANTS WITH BAC
In-Vitro research confirms the development of residual bacterial inhibition zones of up to 7mm, when challenged with Actinomyces viscosus.

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
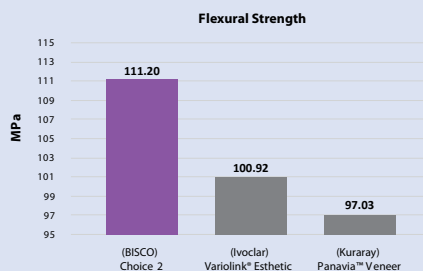
CHOICE™ 2 KIT with all bond Universal Light Cured Veneer Cement

The Choice 2 kit includes everything needed to ensure successful veneer cementation: Proper veneer preparation is critical to achieve optimal bonding. Preparing the veneer requires a proper etch, an application of silane, a thin coat of bonding resin and the CHOICE 2 Veneer Cement.

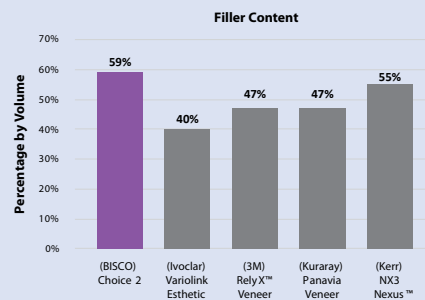Cements with high flexural strength play a crucial role in maintaining the integrity of veneers over time. By providing enhanced support to the restoration, Choice 2 helps to prevent debonding that could compromise the veneers.

Cements with a high filler content provide reinforcement and strength for the restoration to withstand the biting and chewing forces that veneers endure regularly.

CHOICE™ 2 KIT with all bond 3

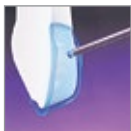
Light Cured Veneer Cement


Choice 2, the Perfect Choice

For Color Stable and Extremely Strong Veneers

PORCELAIN VENEER RESTORATION PREPARATION AND CEMENTATION TECHNIQUE

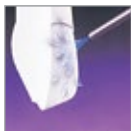
Clean preparation, rinse, dry and then etch


Rinse

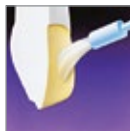Air dry. Leave visibly moist

Mix and apply ALL-BOND 3® Parts A & B

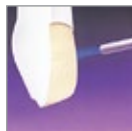
Air dry

Light cure

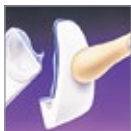
Apply PORCELAIN BONDING RESIN

Line veneer with CHOICE 2

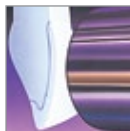Working in pairs, gently seat one veneer at a time

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Choice 2 Shades

A1 A2 A3 B1 Milky Bright Milky Opaque Universal Opaque Translucent

CHOICE 2 is a light-cured luting cement designed specifically for cementation of porcelain and composite veneers. CHOICE 2 exhibits color stability which is a critical factor in aesthetic veneer cementation. The light-cured paste allows dental professionals to maintain total control during placement.

Low Film Thickness

Low film thickness ensures veneers are completely seated.

Highly filled resin cement

Highly filled resin cement enhances the overall strength of the restorations

Ideal for Veneers

Ideal for all veneer restorations.

No Color Shifting

Choice 2 is specifically formulated for color stability resulting in high esthetics.

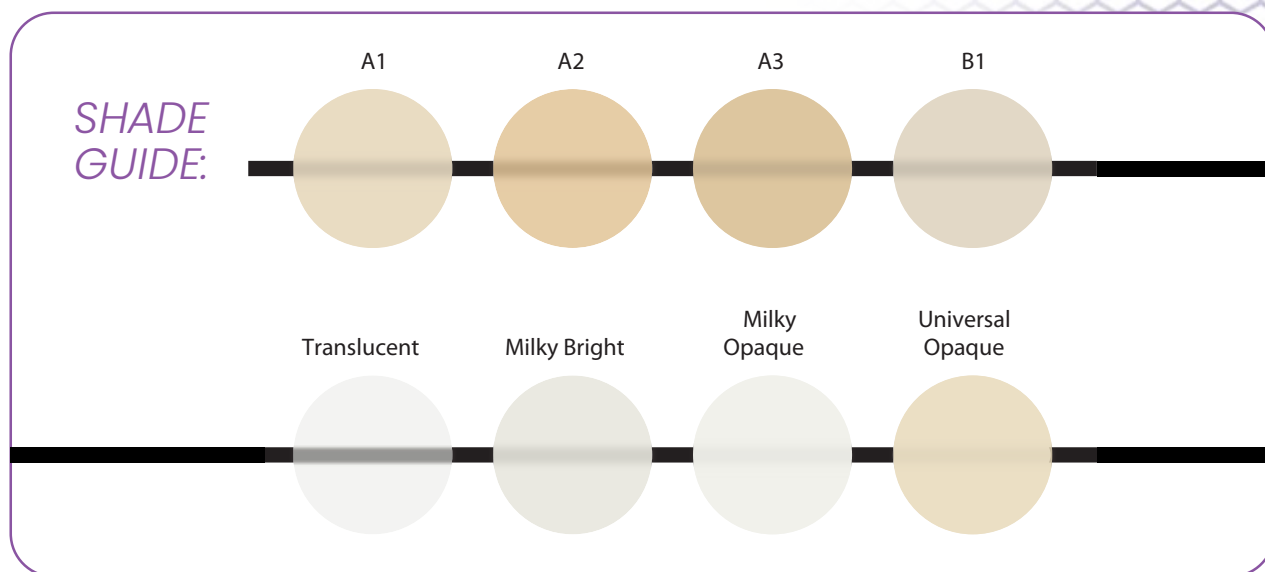
CHOICE™ 2 Try in Paste

Try In Paste 2 Shades


A1 A2 A3 B1 Milky Bright Milky Opaque Universal Opaque Translucent

Corresponding try-in pastes for facilitating shade selection prior to final cementation.


VENEER CEMENTATION GUIDE

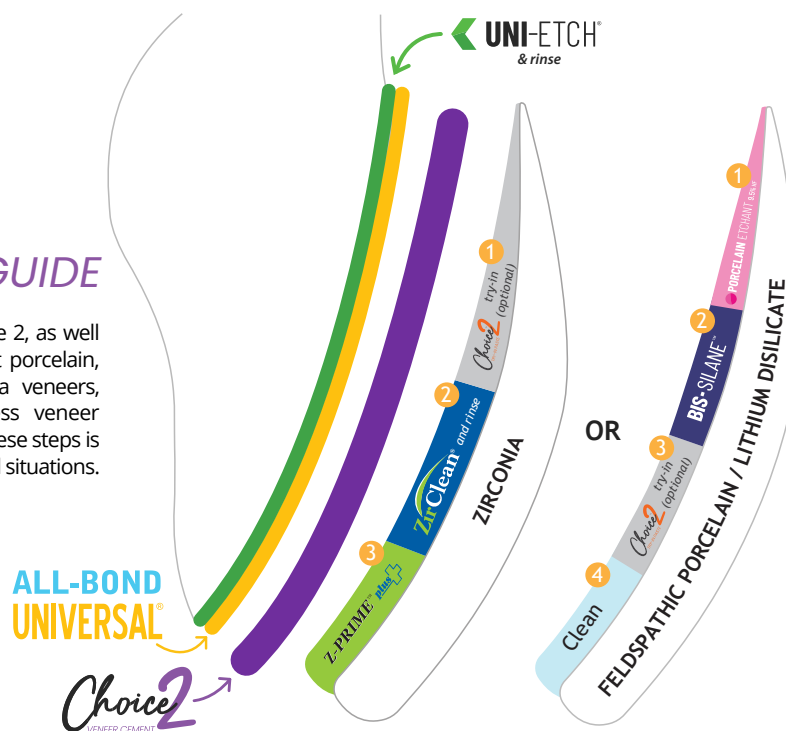Check out all of the ways to utilize Choice 2! Choice 2, as well as all of the essential products needed to cement porcelain, lithium disilicate (e.max), composite, and zirconia veneers, is showcased as a guide for achieving flawless veneer cementation. Using these products and following these steps is a great choice to achieve esthetic results in all clinical situations.


CHOICE™ 2 Veneer Cementation

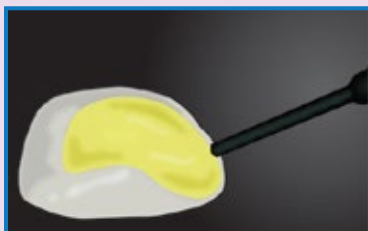using Choice™ 2

OR

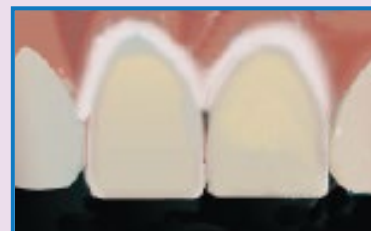
1. Prepare the internal surface of the restoration as instructed by the laboratory:

a. Porcelain/Lithium Disilicate:
If needed, etch the veneer with hydrofluoric acid 9.5% Porcelain Etchant according to the manufacturer's instructions.

b. Apply a thin coat of silane (Porcelain Primer/Bis-Silane™) to the internal surface. Wait for 30 seconds, or dry with (warm) air.

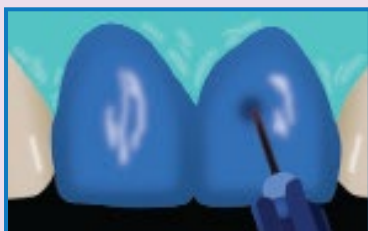
2. Try in the restorations using the corresponding shade of water-soluble try-in paste (Choice™ 2 Try-In Paste). Remove the veneers and either clean with etchant (Uni-Etch® w/BAC or Select HV® Etch w/BAC) and rinse thoroughly, or ultrasonicate in water or alcohol for 2-3 minutes. Dry the restorations.

3. Isolate. Etch preparations (Uni-Etch w/BAC or Select HV Etch w/BAC) for 15 seconds, rinse thoroughly.

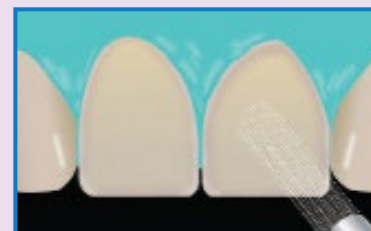4. Dispense adhesive (All-Bond Universal®) in a mixing well. Apply 2 separate coats, agitating each coat for 10-15 seconds.

5. Gently air dry until there is no visible movement of the adhesive. Then thoroughly air dry with greater air pressure. The surface should appear shiny, otherwise apply additional coats.

6. Light cure for 10 seconds.

7. Apply a thin layer of HEMA-free resin (Porcelain Bonding Resin) to the internal surface of the veneer. Do NOT light cure.

8. Using a light-cured resin cement (Choice 2), fill the internal surface of the restoration with the cement.

9. Seat with gentle, passive pressure and light cure for 2-3 seconds to tack the veneer into place. Remove excess cement, then light cure each veneer for 40 seconds.

BIS-SILANE™

BISCO's porcelain primer are silane coupling agents used to improve bonding between porcelain restorations and resin cement. It also protects porcelain restorations from contamination and increases mechanical and chemical bonding of resin to the porcelain. and yields greater resistance to water leakage at the bonding interface.

A two-part silane coupling agent offering additional shelf-life stability to ensure long lasting effective bonding to porcelain

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Versatility

Can be used with all glass ceramic and porcelain substrates.

Porcelain Restorations

Ideal for all porcelain restorations.

Increased Bond Strength

Increases bond strengths to all porcelain and glass -ceramic restorations.

Strong Bond to Lithium Disilicate

Increases bond strengths to all lithium disilicate (e.max®) restorations.

PORCELAIN ETCHANT

BISCO's PORCELAIN ETCHANT improves the adhesive bond between porcelain and cement by increasing the surface area of the porcelain to be bonded. The effect is similar to sandblasting but is more controlled and effective. PORCELAIN ETCHANT will microscopically roughen the feldspathic porcelain or glass surface. BISCO's PORCELAIN ETCHANT is available in 2 concentrations: 4% or 9.5% (Hydrofluoric Acid Gel).

Increased Bond Strength

Improves the mechanical bond between porcelain and adhesive/cement.

Porcelain Restorations

Indicated for cementation of porcelain restorations and repair of porcelain restorations.

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Z-PRIME™ PLUS

Zirconia - Alumina - Metal Primer

BISCO's Z-Prime Plus is a single-component primer used to enhance adhesion between indirect restorative materials and composite resin cements. Z-Prime Plus significantly enhances bond strengths to zirconia, alumina and metal substrates due to its unique combination of two active monomers, MDP, a phosphate monomer, and BPDM, a carboxylate monomer. The combination of adhesive monomers gives Z-Prime Plus a synergistic effect, resulting in its high bond strengths.

High Bond Strength

To zirconia, alumina, and metal restorations

Compatibility

Compatible with light-cured and dual-cured resin luting cements

Single Bottle

Convenient single bottle delivery
Offers ease of dispensing

Versatile

Can be used with all metal and metal-oxide substrates

Enhances Bond Strength

Significantly enhances bond strengths of other resin cements

MDP

Contains MDP, allowing for a strong bond to zirconia, metal, and alumina substrates

Z-Prime Plus Case

1.

Prepared multiple-unit bridge.

2.

After cleaning internal surface with ZirClean®, apply a thin layer of Z-Prime Plus to the zirconia bridge. Wait for 30 seconds and air dry.

3.

Apply two separate coats of All-Bond Universal®, scrubbing the preparation with a microbrush for 10-15 seconds per coat. Do not light cure between coats.

4.

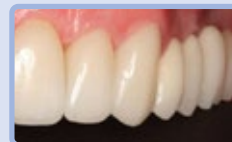
Air dry to remove solvent and light-cure for 10 seconds.

5.

Fill internal surface of the restoration with Duo-Link Universal™ resin cement and immediately seat

6.

Final restoration.

Zir Clean™

Restoration Cleaner

ZirClean is a cleaning gel designed for the non-abrasive cleaning of the bonding surfaces of zirconia (and other prosthetic restorations) after intraoral try-in. ZirClean helps to achieve reliable adhesive cementation results by removing the phosphate contamination of zirconia (as well as ceramic and metal restoration surfaces) that occur during try-in.

Easy to Use

Syringe delivery system offers improved ease of use

Easy Clean up

Easy clean-up of the material

Unparalleled Strength

Helps achieve reliable adhesive cementation results

Contaminant Remover

Contaminants such as saliva can be removed to help achieve better bond strengths after try-in

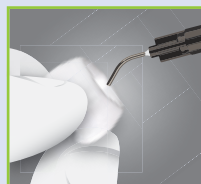
Reliable Results


Helps to achieve reliable cementation results with zirconia restorations after intraoral try-in

1. Try in (to check occlusion and fitting).
2. After try-in, thoroughly rinse the restoration with water spray and dry with oil-free air.

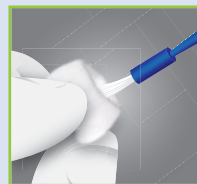3. Cover all bonded surfaces of the restoration with a layer of ZirClean
4. Allow 20 seconds for the cleaning action of ZirClean to take effect, then thoroughly rinse with water spray and dry with oil-free air.

5. Prime the bonding surface of the restoration with a suitable primer (such as Z-PRIME™ Plus or PORCELAIN PRIMER)

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THERACEM®

Bioactive Self-Adhesive Dual Cured Resin Cement

TheraCem is a dual-cured, calcium- and fluoride-releasing, self-adhesive resin cement indicated for luting crowns, bridges, inlays, onlays and posts (prefabricated metal/non-metal/fiber posts).

Calcium and Fluoride Release

TheraCem offers continuous calcium and fluoride release

Alkaline PH

TheraCem transitions from acidic to alkaline pH in minutes

High Degree of Conversion

A high degree of conversion ensures a higher physical strength

Radiopaque

Visible on radiograph to easily distinguish from caries

Easy Clean-Up

Specially formulated to allow for quick and easy clean-up

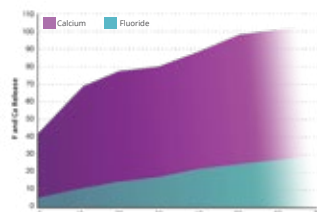
MDP


TheraCem contains MDP offering a strong bond to zirconia, metal, and alumina substrates with no priming or etching required

TheraCem Continuous Fluoride and Calcium Release

Shear Bond Strength to Zirconia

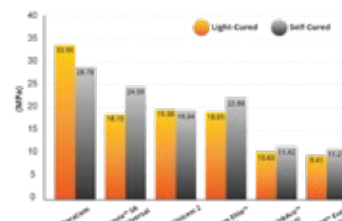TheraCem Case

1. Prepare tooth.

2. Seat restoration with TheraCem.

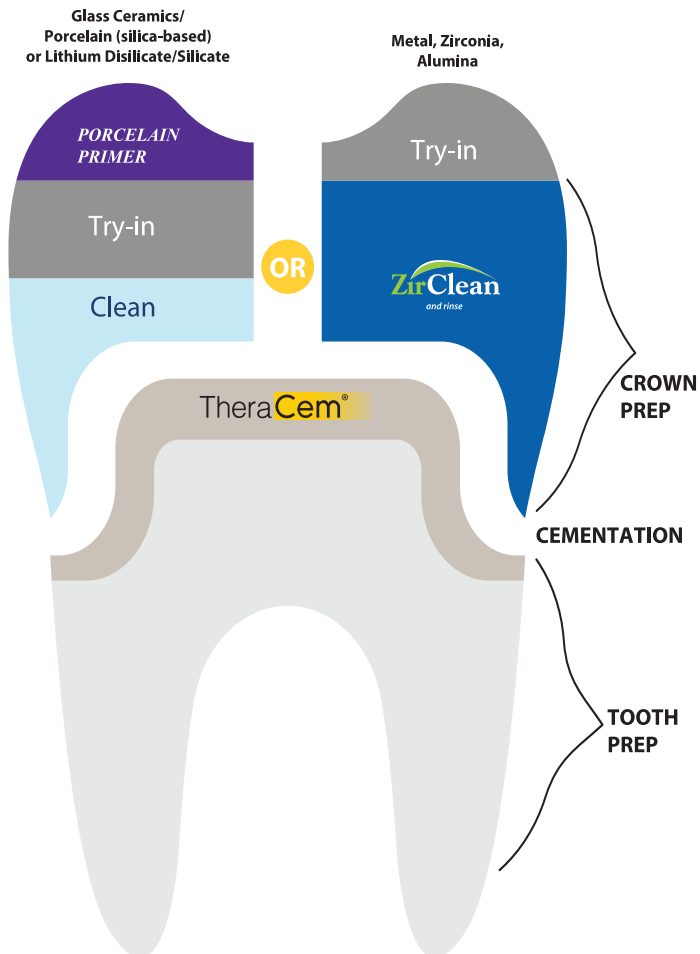
3. Remove excess cement.

4. Inspect margins.

Cementation Without a Bonding Agent Technique

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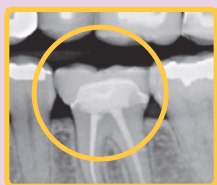
DUO-LINK UNIVERSAL™ Resin Luting Cement

Highly radiopaque
with easy clean-up

Duo-Link Universal is specially formulated for cementation of ALL indirect restorations. The adhesive resin cement is intended for use with adhesives designed for compatibility with all dental materials, including all BISCO adhesives.

Easy Clean-Up

Formulated to allow for quick and easy removal of excess cement

Easy to Use

Auto-mix, dual-syringe provides a consistent mix and easy placement

Radiopaque

Visible on radiograph to easily distinguish from caries

CAD/CAM Restorations

Ideal for all chairside and lab-fabricated restorations

High Degree of Conversion

In both light- and self-cured modes ensures a strong, long lasting restoration

Universal For All Cementation Procedures

Crowns, bridges (fixed prosthesis), inlays, onlays, and posts/dowels, fabricated from metal, composite, porcelain, ceramic, zirconia, alumina, CAD/CAM restorations, etc.

Duo-Link Case

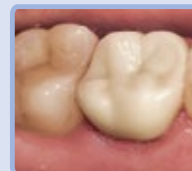
1. Duo-Link Universal dispensed in the crown

2. All-Bond Universal applied, air dried, and light-cured


3. Restoration seated. Excess cement removed

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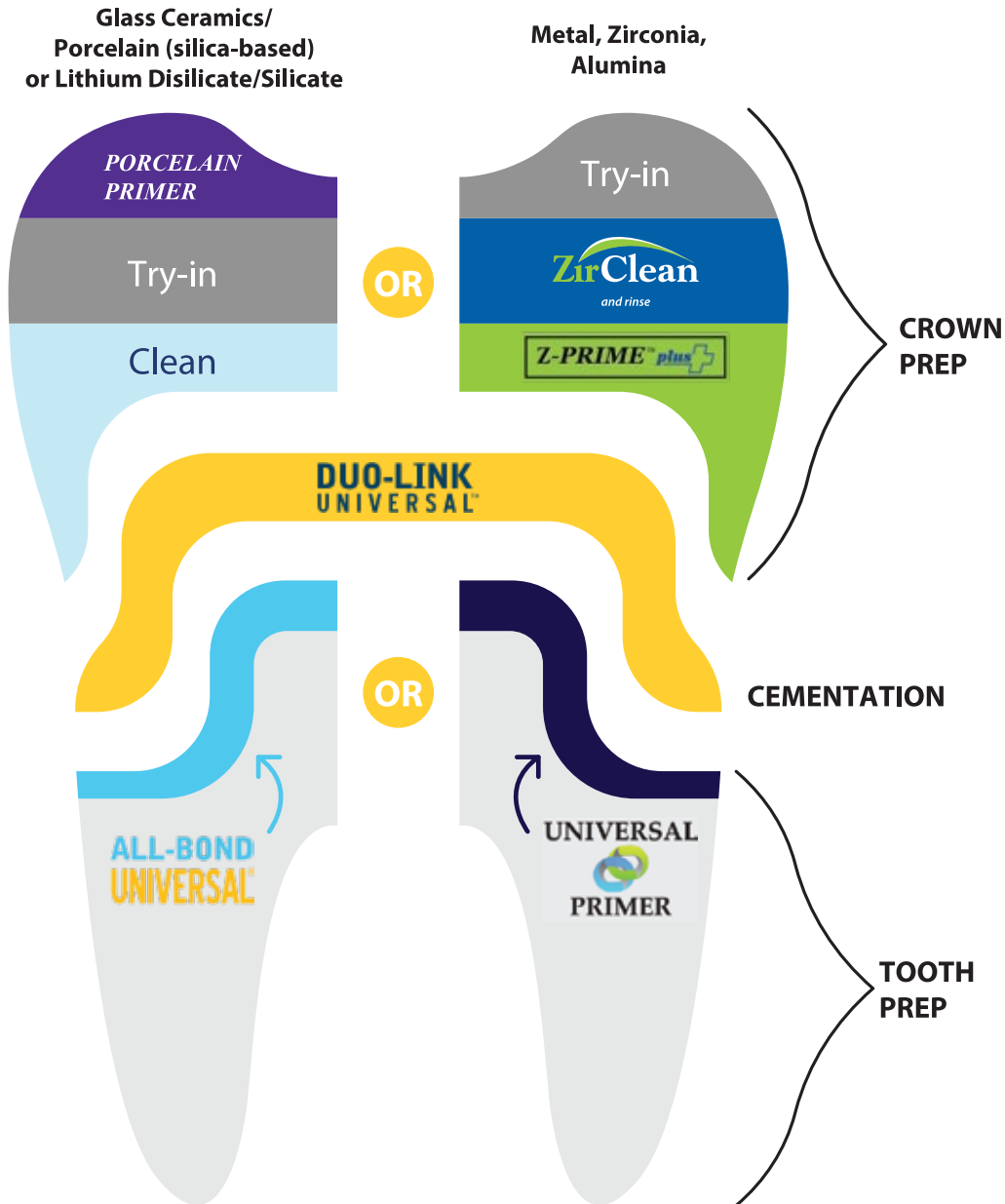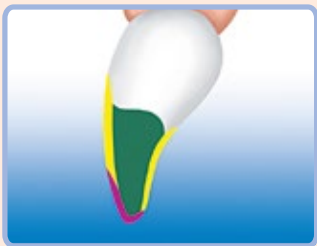
AELITE™ AESTHETIC ENAMEL

Light-Cured Reinforced Nanofil Composite

AESTHETIC ENAMEL SHADES

A1	A2	A3	A3.5	B1	BLEACH W-E
----	----	----	------	----	------------

INCISAL SHADES

FROST	CLEAR
-------	-------

Aelite Aesthetic Enamel is a light-cured, highly reinforced, nanofil composite with a complete range of VITA® shades as well as three incisal shades. Aelite Aesthetic Enamel has high strength and can be placed as the enamel layer in both posterior and anterior restorations. Aelite Aesthetic Enamel has excellent polishability due to the nano-sized filler particles, making it suitable and desirable for anterior restorations. The reinforcing filler gives Aelite Aesthetic Enamel maximum strength.

Light Cured

Light-cured composite

Easy Handling

Superior handling properties allow for easy placement, contouring, and sculpting.

High Flexural Strength

Endures load in stress bearing areas due to high flexural strength.

Radiopaque

Radiopaque for easy detection on radiographs.

High Compressive Strength

High compressive strength offers maximum durability.

High Wear Resistance

High filler loading (88% by weight), ensures composite will not prematurely wear by maximizing marginal integrity and stain resistance.

Highly Esthetic

Available in a variety of VITA shades that provide natural translucency and optimal color matching with adjacent dentition.

Low Shrinkage

Low volumetric shrinkage allows for minimized stress.

High Gloss Finish

Exceptional polishability allows the clinician to create the most esthetic anterior and posterior restorations that polish to the highest luster.

AELITE™ ALL-PURPOSE BODY

Light-Cured Universal Microhybrid Composite

ALL-PURPOSE BODY SHADES

A1

A2

A3

A3.5

B1

Aelite All-Purpose Body is a durable, light-cured, universal microhybrid composite suitable for all applications from Class I through Class VI. Its high strength makes it ideal to form the core of layered esthetic restorations. Aelite All-Purpose Body can be used as part of a layering technique or stand-alone as a universal restorative. This composite features a complete line of body shades and two opaque shades.

Light Cured

light-cured composite

Easy Handling

Excellent handling allows greater ability to contour and sculpt final restorations.

High Flexural Strength

Endures load in stress bearing areas due to high flexural strength (128 MPa).

Radiopaque

Radiopaque for easy detection on radiographs.

High Compressive Strength

High compressive strength (312 MPa) makes composite strong enough to be placed in any class.

Durable

High strength makes material ideal to form the core of layered aesthetic restorations.

Highly Esthetic

Designed to blend with surrounding dentition when used alone or in the layering technique with Aelite Aesthetic Enamel.

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AELITE™ LS POSTERIOR

Light-Cured Low Shrink Hybrid Composite

POSTERIOR DENTIN (PLS) SHADES

A1

A2

A3

A3.5

Aelite LS Posterior is a light-cured, highly filled, low shrinkage posterior composite that helps to reduce contraction stress to overcome stress related problems. Due to the high filler content, handling of this composite simplifies establishment of occlusal contours, proximal surfaces and interproximal contacts.

Light Cured

light-cured composite

Easy Handling

BISCO's proprietary filler enhances handling properties, featuring higher control for sculptability with minimal tackiness and pull-back.

High Flexural Strength

Endures load in stress bearing areas due to high flexural strength (120 MPa).

High Gloss Finish

High filler loading (88% by weight) provides exceptional polishing luster.

High Compressive Strength

High compressive strength (261 MPa) makes this composite an ideal choice for posterior restorations.

High Wear Resistance

High filler loading (88% by weight), ensures composite will not prematurely wear by maximizing marginal integrity and stain resistance.

Radiopaque

Radiopaque for easy detection on radiographs.

Highly Esthetic

Available in a variety of VITA shades that provide natural translucency and optimal color matching with adjacent dentition.

Low Shrinkage

Low shrinkage provides long-term patient satisfaction

AELITEFLO™

Low Modulus Microhybrid Composite

AELITEFLO is a light-cured composite with flow characteristics which make it ideal for Class V restorations, as well as Class III's and small Class IV's. Other uses of this material are as a liner for Class I and II restorations, pit and fissure sealants, the repair of marginal defects, small core build-ups, blocking out undercuts, porcelain repair, porcelain veneer cementation or as a flowable composite for bonding fibrous splints into place.

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Light-cured composite

Easy Handling

Extremely easy handling for excellent adaptation to dentin.

Flowable

Flowable viscosity allows composite to be easily dispensed and injected into the preparation without dripping or slumping.

Elastic properties

Elastic properties lead to better retention.

High Compressive Strength

Superior compressive strength at 266 MPa.

Versatile

Ideal for Class V, Class III, and small Class IV restorations. Aeliteflo can also be used as a liner for Class I and II restorations, pit and fissure sealant, small core build-ups, and more.

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MODELING RESIN

Sculpting and Wetting resin Agent for Composite

Modeling Resin is a light-curable, low viscosity microfilled resin (30% by weight) designed for use as a composite sculpting resin. By wetting your instrument with Modeling Resin, you can facilitate the placement and sculpting of all direct restorations. Unlike adhesives, Modeling Resin contains no solvent and is HEMA-free, so it will enhance composite placement without weakening the restoration.

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Quantium™

Universal Composite

Power Up Your Dentistry, Layer by Layer

NEW

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Quantium is a light-cured, radiopaque universal composite developed with an optimized filler blend to deliver excellent handling, exceptional strength, low polymerization shrinkage, and natural esthetics. Designed for both single-shade and layered techniques, it provides durable, predictable results for direct and indirect restorations in anterior and posterior applications.

Durable Restorations

High strength, wear resistance, and low shrinkage create durable restorations

Low Shrinkage

Features low volumetric shrinkage, which minimizes microleakage and ensures marginal adaptation

Excellent Esthetics

Easily polished and stain resistant to create esthetic results that maintain their appearance over time

High Radiopacity

Easy identification on radiographs aiding in post-treatment evaluation

Easy to Handle

Smooth delivery and non-sticky handling

Quantum A3 Enamel

Radiograph

Indications:

- Direct anterior and posterior restorations
- Core build-up material
- Splinting
- Indirect restorations including inlays, onlays, and veneers

Compressive Strength
308.39 MPa

Flexural Strength
120.933 MPa

Volumetric Shrinkage
2.28%

Diametral Tensile Strength
40.30 MPa

QUANTIUM IN ACTION

1 Pre-op case of old amalgam restoration.

2 The preparation was cleaned with a slurry of pumice and water, rinsed and dried.

3 TheraBase was applied to the dentin surface of the prepared cavity and light-cured for 20 seconds.

4 Select HV Etch was used in a selective-etch bonding technique.

5 All-Bond Universal was applied, following manufacturer's instructions.

6 Quantum A2 Body and Enamel were used in a layering technique.

7 Final restoration.

BEAUTIFUL RESTORATIONS AT YOUR FINGERTIPS!

Quantum makes shade selection simple with a broad range of options designed to match the VITA®+ Classical Shade Guide, reducing guesswork and helping you achieve natural-looking results.

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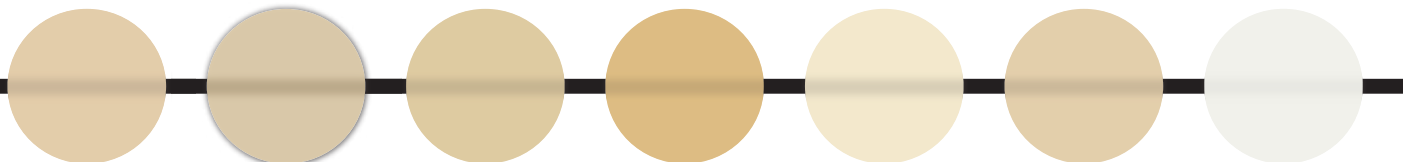
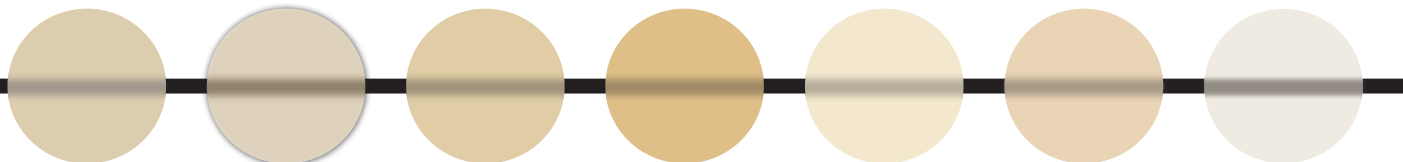
CORE-FLO™ DC

Dual-cured Flowable

Core Build-Up/Dentin Replacement Material

Dispenser
Required

Core-Flo DC Shades:

Natural/A1

Opaque White

Core-Flo™ DC is a versatile dual-cured core material dispensed by a 1:1 syringe dispenser. It is ideal for core build-ups, post cementation and dentin replacement. Core-Flo DC is radiopaque and available in two esthetic shades, Natural/A1 and Opaque White, to meet all of your direct and indirect restorative requirements.

High Compressive Strength

LC = Nat: 269, Opaque: 273 Mpa
SC = Nat: 259, Opaque: 256 Mpa

Radiopaque

1.9mm
Easy to distinguish

Stackable (Core-Flo DC)

Optimal handling when building up core

High Flexural Strength

LC = Nat: 134, Opaque: 113 Mpa
SC = Nat: 113, Opaque: 120 Mpa

Cuts Like Dentin

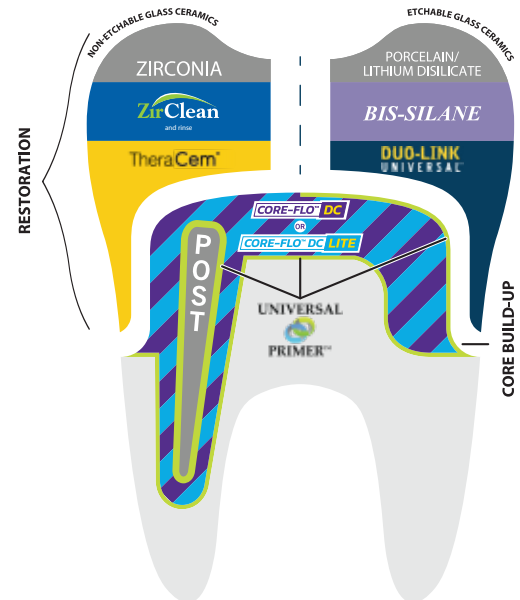
Allows for easy preparation

Working Time/ Setting Time

2 Minutes MIN. 7 Minutes MAX.
At Room temp.

Post Cementation With Core Build-up Technique

CORE-FLO™ DC Lite

Self-Leveling Dual-Cured Core Material

Core-Flo DC Lite

Core-Flo DC Lite Shades:

Natural/A1

Opaque White

Core-Flo™ DC Lite is a dual-cured core material that is dispensed with an auto-mix dual syringe. Core-Flo DC Lite's optimal self-leveling viscosity allows for excellent adaption resulting in gap-free margins when replacing natural dentition with a direct core build-up. In addition, the clinician can cement the post and build-up the core. available in Natural/A1 and Opaque White shades.

High Compressive Strength

LC = Nat: 258, Opaque: 233 Mpa
SC = Nat: 205, Opaque: 215 Mpa

High Flexural Strength

LC = Nat: 121, Opaque: 114 Mpa
SC = Nat: 125, Opaque: 171 Mpa

Radiopaque

1.5mm
Easy to distinguish

Cuts Like Dentin

Allows for easy preparation

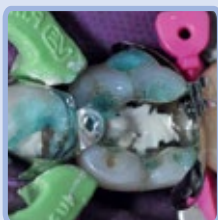
Self-Leveling (Core-Flo DC Lite)

Allows for excellent adaption resulting in gap-free margins

Working Time/ Setting Time

1 Minutes MIN/ 7 Minutes MAX.
At Room temp.

1. Enamel etched with Select HV Etch

2. Core-Flo DC applied after use of Universal Primer™

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

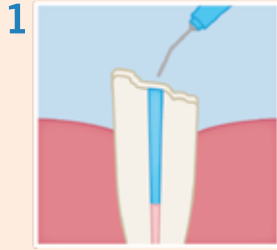
POST CEMENTATION & CORE BUILD-UP

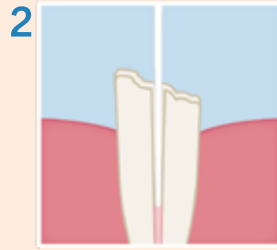
If using the TOTAL-ETCH TECHNIQUE, etch the canal with Uni-Etch® or Select HV® Etch for 15 seconds. If using the SELF-ETCH TECHNIQUE, proceed to the next step.

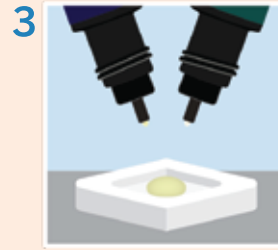Clean and irrigate. Rinse canal with water. Remove excess water with a brief burst of air and paper point(s) to remove the pooled water from the canal.

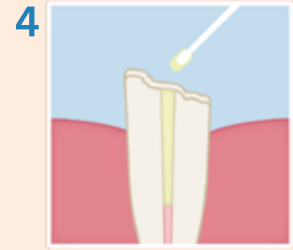
Dispense Universal Primer™ Mix adhesive for 5 seconds.


With an endodontic brush, apply and scrub 2 separate coats of Universal Primer into the canal. Remove excess pooling of material with paper points and light suction. Gently air dry.

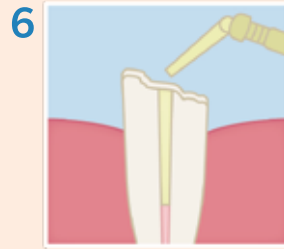
Coat the post with Universal Primer. Air Dry.

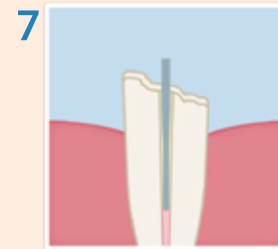
Inject Core-Flo™ DC or Core-Flo™ DC Lite into the canal.

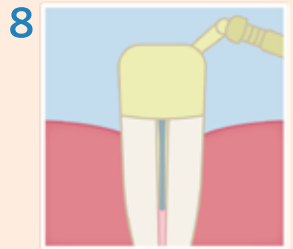Coat the apical end of the post with Core-Flo DC or Core-Flo DC Lite. Seat the post into the canal gently. Maintain firm pressure for 20-30 seconds.

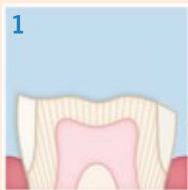
Continue to express Core-Flo DC or Core-Flo DC Lite around the post and build up the core to the desired height and shape. Remove the excess material and light cure for 40 seconds.


Isolate tooth and prepare accordingly. Rinse and dry.

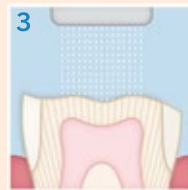Dispense 1-2 drops of Universal-Primer™ A & B (or any BISCO adhesive) into a clean well. Apply two separate coats of Universal Primer, scrubbing the preparation with a microbrush for 10-15 seconds per coat. Do not Light Cure between coats.

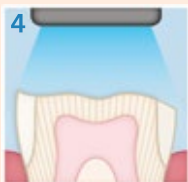
Evaporate excess solvent by thoroughly air-drying with an air syringe for at least 10 seconds; there should be no visible movement of the adhesive. The surface should have a uniform glossy appearance; otherwise, apply an additional coat of Universal Primer and air dry.

Light cure for 10 seconds.

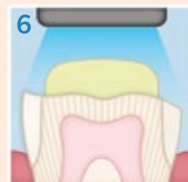Place the end of the intra-oral tip directly on to the preparation, and express CORE-FLO™ DC or CORE-FLO™ DC Lite.

Light cure the Core-Flo DC or CORE-FLO DC Lite for 40 seconds and continue with final preparation

CORE BUILD-UP TECHNIQUE

THERACAL PT

Dual-Cured Resin Modified Calcium Silicate Pulpotomy Treatment

TheraCal PT

TheraCal PT is a biocompatible, dual-cured, resin-modified calcium silicate designed for pulpotomy treatment. TheraCal PT maintains tooth vitality by performing as a barrier and protectant of the dental pulpal complex.

Calcium Release

Unique hydrophilic matrix facilitates calcium release

Easy Syringe Application

Manual mixing is not required, the dispensing tip creates a uniform mix allowing for direct placement

Dual Cured

One-layer confidence

Radiopaque

TheraCal PT is radiopaque (2.45mm) allowing for easy identification and differentiation from recurrent decay and other restorative materials.

Protective Liner

Suitable as a protective liner or base under a variety of substrates.

Alkaline pH

Alkaline pH at 7 days.

Working Time/ Setting Time

45 seconds MIN/5 Minutes MAX.
At 35°C.

Moisture Tolerant

Unlike calcium hydroxide, TheraCal PT has low solubility and will not wash out over time.

Pulpotomy Technique

1. Perform pulpotomy treatment.

2. Achieve hemostasis

3. Place TheraCal PT directly in the pulp chamber, and ensure good adaptation to the cavity walls and margins.

4. Light cure TheraCal PT for 10 seconds. Place desired adhesive, base, and/or restoration following manufacturer's directions for restoring tooth.

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THERACAL® LC

Resin-Modified Calcium Silicate Pulp Protectant/Liner

TheraCal LC is a light-cured resin modified calcium silicate ideal for direct and indirect pulp capping and as a protective liner.

Pulp Capping Agent

Designed for use in direct and indirect pulp capping.

Protective Seal

Significant calcium release leads to protective seal

Insulates Pulp

Forms a protective barrier that insulates the pulp

Radiopaque

Visible on radiograph to easily distinguish from caries

LightCured

Light cured material, which permits immediate placement of restorative material.

Moisture Tolerant

Has low solubility and will not wash out over time

Protective Liner

Performs as an ideal protective liner under composites, amalgams, cements, and other base materials.

Alkaline pH

Alkaline pH promotes pulp vitality

Versatility

Can be used with all etch techniques (self-, selective-, and total-etch) for optimal bonding and finishing of the restoration.

Time Saver

Can be directly applied to the pulp exposure and be used as a liner in the same prep, with no need for two separate products.

Calcium Release

Stimulates hydroxyapatite formation and secondary dentin bridge formation

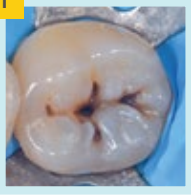
Easy Syringe Application

Easy and fast syringe delivery offers controlled and precise placement. Material is easy to manipulate without dripping or slumping.

THERACAL[®] LC

Indirect Pulp Capping Placement Dentistry courtesy of Dr. Juan Angel Castro

1

Isolate tooth.

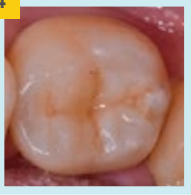
2

Remove infected carious tooth structure. Leave preparation visibly moist.

3

Apply TheraCal LC directly to visibly moist dentin. Layer is not to exceed 1mm in depth. Manipulate into a smooth surface. Light-cure for 20 seconds.

4

Continue tooth restoration.

Direct Pulp Capping Placement Dentistry courtesy of Dr. Elvio Durando

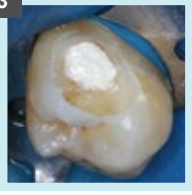
1

Isolate tooth.

2

Remove infected carious tooth structure. Achieve hemostasis. Leave preparation visibly moist.

3

Apply TheraCal LC directly to exposed pulp. Layer is not to exceed 1 mm in depth. Cover all the exposed areas and extend TheraCal LC at least 1 mm onto sound dentin surrounding the exposure. Light-cure for 20 seconds.

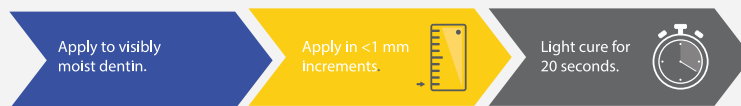
4

Apply adhesive. (All-Bond Universal with selective-etch technique shown)


Continue tooth restoration.


TheraCal LC Quick Tips

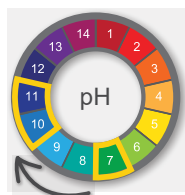Calcium Release
TheraCal LC releases 213 µg/cm² of calcium in 24 hours.

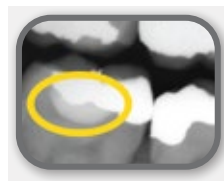Alkaline pH
TheraCal LC reaches an Alkaline pH of 10-11 in 3 hours.

Radiopaque
TheraCal LC is radiopaque for easy identification on radiographs.

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Thera®Base

Dual-Cured Self-Adhesive Calcium Releasig Base/Liner

TheraBase is a dual-cure, calcium and fluoride releasing, self-adhesive base/liner. Utilizing the THERA technology, TheraBase chemically bonds to tooth structure, and releases and recharges calcium and fluoride ions. TheraBase's calcium release generates an alkaline pH which promotes pulp vitality. It is a dual-cured material that will polymerize even in deep restorations where light cannot reach.

TheraBase is stronger than other base materials, glass ionomers and resin-modified glass ionomers. Additionally, it is radiopaque allowing for easy identification on radiographs, providing a quick and effective diagnosis.

Releases Calcium and Fluoride

continuous release of calcium and Fluoride ions.

Self Adhesive

No bonding agents required
Save time and money

High Flexural Strength

Stronger and more fracture resistant

High compressive Strength

Absorbs shock and stress from occlusal forces without fracturing

Contains MDP

Contains the adhesion promoting monomer MDP, ensuring reliable and optimal bond to dentin.

Easy to Use

Auto-mix, dual-syringe provides a consistent mix for immediate delivery with zero to minimal waste of material

Dual-Cured

Material will fully cure even in deep restorations where light cannot reach

Alkaline pH

Generates an Alkaline pH (pH=11*) in minutes, which promotes pulp vitality.

High Degree of Conversion

A high degree of conversion ensures enhanced physical properties.

Radiopaque

TheraBase is radiopaque allowing for identification on radiographs and effective diagnosis.

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1 After cavity preparation, all water was removed using a stream of air, leaving the surface visibly moist. TheraCal LC was applied on small pulp exposure and light-cured for 20 seconds.

2 TheraBase was applied to the dentin surface of the prepared cavity directly from the dispensing syringe.

3 TheraBase was light cured for 20 seconds. If desired, TheraBase can be allowed to self-cure for 4 minutes.

4 A selective-etch bonding technique was used to condition the surface of the preparation. Any bonding technique can be applied.

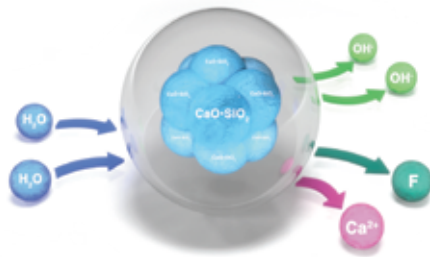
5 All-Bond Universal[®] was applied following manufacturer's instructions.

6 Restorations were filled with a light-cure composite material following manufacturer's instructions.

THERA HYDROPHILIC MATRIX


BISCO has developed a hydrophilic matrix that allows for ion exchange. Previous or traditional resin matrices have been hydrophobic, but BISCO's matrix allows for ion exchange as water goes into the matrix, reacts, and calcium hydroxide ions and fluoride ions are released.


At the calcium-rich interface between TheraBase and the dentin, an alkaline pH environment is created.

NO Pulp Exposure

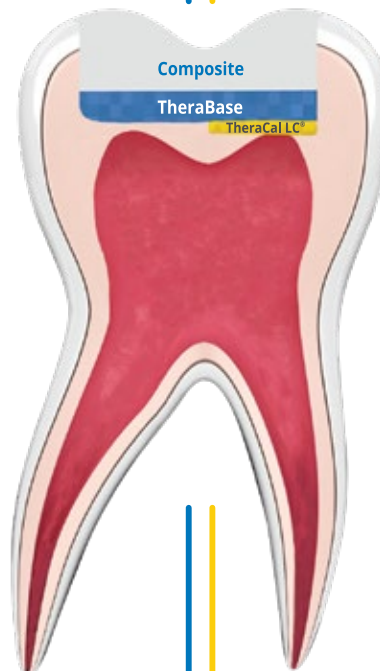Pulp Exposure

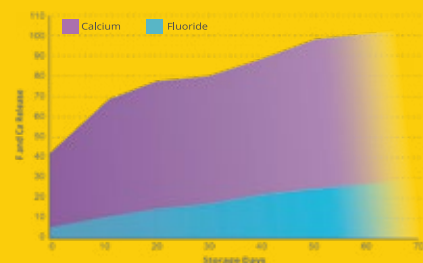TheraBase Continuous Fluoride and Calcium Release¹

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